### Calibrador Lp(a) Monlabtest®

Turbidimetría

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### Lp(a) Calibrator Monlabtest®

Turbidimetry

#### Quantitative determination of Lipoprotein (a) (Lp(a))

**IVD** Only for professional in vitro diagnostic use  
Store at 2-8°C

#### PRODUCT CHARACTERISTICS

The calibrator is a lyophilised serum from human origin containing a Lp(a) concentration suitable to calibrate Lp(a) determinations by turbidimetry method.

#### REAGENTS

<b>Calibrator-Lp(a)</b>	Calibrator. Human serum. Sodium azide 0.95 g/L. Lp(a) concentration is stated on the vial label.
-------------------------	--

#### PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

#### CALIBRATION

The concentration value of the Calibrator has been standardized against an Internal Reference Material.  
Reference Method: ELISA method stated in *J. Lipid Research 30, 651-662, 1989*.

#### PREPARATION

Reconstitute (→) with 1.0 mL of distilled water. Mix thoroughly, avoiding foam forming. Bring to room temperature for about 10 minutes before use.

#### STORAGE AND STABILITY

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

After reconstitution, stable for 1 month at 2-8°C or 3 months at -20°C.

#### PROCEDURE

To be used in turbidimetric assays.

#### PRESENTATION

MO-165051 1 x 1 mL