FR Calibrador Monlabtest® Turbidimetría	 
---	--

RF Calibrator Monlabtest® Turbidimetry	 
--	--

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Quantitative determination of Rheumatoid Factors (RF)

Para uso profesional de diagnóstico in vitro
 Conservar a 2-8°C

Only for professional in vitro diagnostic use
 Store at 2-8°C

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PRODUCT CHARACTERISTICS

The calibrator is a lyophilised serum from human origin containing a RF concentration suitable to calibrate RF determinations by turbidimetry method.

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REAGENTS

RF Calibrator	Calibrator. Human serum. RF concentration is stated on the vial label.
----------------------	--

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PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

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CALIBRATION

The concentration value of the Calibrator has been standardized against the International Reference Standard from NIBSC 64/002.

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PREPARATION

Reconstitute (→) with 2,0 mL of distilled water. Mix thoroughly, avoiding foam forming. Bring to room temperature for about 10 minutes before use.

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STORAGE AND STABILITY

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

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After reconstitution, stable for 1 month at 2-8°C or 3 months at -20°C.

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