**Control CK-Nac / CK-MB**
**Monlabtest®**

Suero Control Humano

**Suero Control CK/CK-MB**

**IVD** Para uso profesional de diagnóstico in vitro  
 Conservar a 2-8°C

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Human Control Serum

**CK/CK-MB Control Serum**

**IVD** Only for professional in vitro diagnostic use  
 Store at 2-8°C

**PRODUCT CHARACTERISTICS**

CK-Nac / CK-MB CONTROL is a human lyophilised control serum. Is intended for control of accuracy for use with manual and automated analytical procedures.

**REAGENTS**

Human serum. Biological additives. Bacteriostatics agents. The concentration of the components are lot-specific. See chart below for the values of ingredients.

**PRECAUTIONS**

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

**PREPARATION**

-Reconstitute (→) with 2.0 mL of distilled water.  
 -Mix thoroughly, avoiding foam forming.  
 -Bring to room temperature for about 30 min. before use.  
 Improper handling and/or storage can affect results.  
 Inaccurate reconstitution and errors in assay technique can cause erroneous results.

**STORAGE AND STABILITY**

The Control serum is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date or if there is visible evidence of microbial growth.

Store tightly capped when not use.  
 - After reconstitution is stable for:  
 At 15°C to 25°C 24 hours  
 At 2°C to 8°C 3 days  
 At -25°C to -15°C 1 month

Component	Method	T <sup>a</sup>	Value	Range	Units
<b>CK-Nac</b>	DGKC	37 °C			U/L
<b>CK-MB</b>	Immunological Method	37 °C			U/L

**PRESENTATION**

MO-165110 2 x 2 mL