Control Ferritina
Monlabtest®

Turbidimetry

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Ferritin Control
Monlabtest®

Turbidimetry

Quantitative determination of Ferritin

IVD Only for professional in vitro diagnostic use
Store at 2-8°C

PRODUCT CHARACTERISTICS

The control is a lyophilised human serum containing ferritin used to evaluate the precision and accuracy of ferritin determinations in human serum by turbidimetry method.

REAGENTS

Ferritin Control	Human serum. With a low concentration level of ferritin. The value of ferritin is stated on the vial label.
------------------	---

PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

CALIBRATION

The concentration value of the Calibrator has been standardized against the 2nd International Ferritin Standard (85/578) from WHO (1992).

PREPARATION

Reconstitute (→) with 2.0 mL of distilled water. Mix thoroughly, avoiding foam forming. Bring at room temperature for about 10 minutes before use.

STORAGE AND STABILITY

The control is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date. After reconstitution, stable for 10 days at 2-8°C or 6 months at -20°C.

PROCEDURE

To be used in turbidimetric assays.

PRESENTATION

MO-165056 1 x 2 mL