Calibrador Ferritina

Monlabtest®

Turbidimetría

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Monlabtest®

Turbidimetry

Quantitative determination of Ferritin

IVD Only for professional in vitro diagnostic use
Store at 2-8°C

PRODUCT CHARACTERISTICS

The calibrator is a lyophilised serum from human origin containing a ferritin concentration suitable to calibrate ferritin determination by turbidimetry method.

REAGENTS

Ferritin-Calibrator	Calibrator. Human serum. Ferritin concentration is stated on the vial label.
----------------------------	--

PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

CALIBRATION

The concentration value of the Calibrator has been standardized against the 2nd International Ferritin Standard (85/578) from WHO (1992).

PREPARATION

Reconstitute (→) with 3.0 mL of distilled water. Mix thoroughly, avoiding foam forming. Bring to room temperature for about 10 minutes before use.

STORAGE AND STABILITY

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

After reconstitution, stable for 1 month at 2-8°C or 3 months at -20°C.

PROCEDURE

To be used in turbidimetric assays.

PRESENTATION

Cont. MO-165055

1 x 3 mL