<p>Control Ferritina MonlabTest® Turbidimetría</p>	
--	------

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Quantitative determination of Ferritin

Only for professional *in vitro* diagnostic use.
 Store at 2-8°C.

PRODUCT CHARACTERISTICS

The control is a lyophilised serum from human origin containing ferritin used to evaluate the precision and accuracy of ferritin determinations by turbidimetry method.

REAGENTS

Ferritin Control	Human serum. With a low concentration level of ferritin. The value of ferritin is stated on the vial label.
------------------	---

PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

CALIBRATION

The concentration value of the control has been standardized against the 2nd International Ferritin Standard (80/578) from WHO (1992).

PREPARATION

Reconstitute (→) with 2.0 mL of distilled water. Mix thoroughly, avoiding foam forming. Bring at room temperature for about 10 minutes before use.

STORAGE AND STABILITY

The control is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

After reconstitution, stable for 1 month at 2-8°C or 3 months at -20°C

PROCEDURE

To be used in turbidimetric assays.

PRESENTATION

MO-165056	1 x 2 mL
-----------	----------